

PRESSURE CARE

Active Air Mattresses	98
Static Air Mattresses	108
Foam Mattresses	110
Pressure Relief Aids	116
Gel Pads and Cushions	118



Scan to access
Pressure Care
Equipment online





Together with caregivers, eliminate pressure ulcers.

This has been our goal since our start in 1999 – to develop world-class tools for the healthcare sector, making it possible to take excellent care of patients, and improve health economy.
Welcome to take part in our quest.

/Magnus Högberg, founder, owner and CEO

Pressure Ulcer Risk Assessment

Categorisation of pressure ulcers



Category 1

Intact skin with non-blanchable redness of a localized area usually over a bony prominence. Other signs of the area such as discolouration, oedema, warmer or cooler as compared to adjacent tissue may also indicate pressure injury. (Category 1 may be difficult to detect in individuals with dark skin tones. May indicate "at risk" persons.)




Category 2

Damage to the epidermis and/or dermis. The ulcer is superficial and may be visible as a blister or an abrasion of the skin.



Category 3

Involves damage to the full thickness of the skin and includes injury to, or necrosis of, the subcutaneous tissue layer, and may extend down to but not through the underlying fascia.

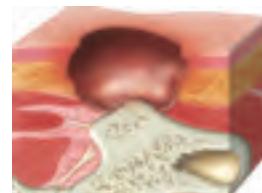

Category 4

Deep tissue damage, necrosis or damage to the muscle, bone or connective tissue with wound cavitations.

Risk assessment according to the Modified Norton scale

A. Mental condition			
4	3	2	1
Fully oriented	Occasionally confused	Cannot answer adequately	No contact
B. Physical activity			
4	3	2	1
Ambulant	Walks with help	Chair bound	Bedridden
C. Mobility			
4	3	2	1
Full	Slightly limited	Very limited	Immobile
D. Food intake			
4	3	2	1
1 portion and/or equivalent	¾ portion and/or equivalent	½ portion and/or equivalent	<½ a portion
E. Fluid intake			
4	3	2	1
>1000ml/24 hours	700 – 1000ml/24 hours	500 – 700ml/24 hours	<500 ml/24hours
F. Incontinence			
4	3	2	1
None	Occasional	Usually urine (catheter)	Urine and feces
G. Physical condition			
4	3	2	1
Good	Fair	Poor	Very poor

Count up the score.

20 points or lower = increased risk of pressure damage. Very active decubitus prophylaxis/tighter overall care!

Ek AC, Unosson M, Bjurulf P. The Modified Norton Scale and the nutritional intake, (1989) Scand J Caring Sci 3:4;183-187.

This modified Norton scale is translated by Anna-Christina Ek, Professor Em, Hälsouniversitetet, Linköping.

National Pressure Ulcer Advisory Panel, European Pressure Ulcer Advisory Panel and Pan Pacific Pressure Injury Alliance. Prevention and Treatment of Pressure Ulcers: Quick Reference Guide. Emily Haesler (Ed). Cambridge Media: Osborne Park, Western Australia; 2014.

Risk assessment according to the Braden scale

A. Sensory perception			
1	2	3	4
Completely limited	Very limited	Slightly limited	No impairment
B. Moisture			
1	2	3	4
Constantly moist	Very moist	Occasionally moist	Rarely moist
C. Activity			
1	2	3	4
Bedfast	Chairfast	Walks occasionally	Walks frequently
D. Mobility			
1	2	3	4
Completely immobile	Very limited	Lightly limited	No limitation
E. Nutrition			
1	2	3	4
Very poor	Probably inadequate	Adequate	Excellent
F. Friction & shear			
1	2	3	
Problem	Potential problem	No apparent problem	

Count up the score.

9 points = Very high risk for pressure ulcer development

10-12 points = High risk for pressure ulcer development

13-14 points = Medium risk for pressure ulcer development

15-18 points = Low risk for pressure ulcer development

© Copyright Barbara Braden & Nancy Bergstrom 1988

Courtesy of Care of Sweden





 care of sweden

CuroCell UNO

A new generation of fully automated alternating pressure mattress replacement systems

The CuroCell UNO is used as an aid in the prevention and treatment of pressure ulcers up to and including category 4. It is equipped with a large touch screen panel with a user-friendly interface. The fully automated pump offers automatic adjustment, providing optimal pressure relief and comfort for each individual user while aiding simple use.

ORDER CODES:

0401-1550: 85 x 200 x 20cm
 0401-1551: 90 x 200 x 20cm
 0401-1552: 105 x 200 x 20cm
 0401-1553: 120 x 200 x 20cm

Features & Benefits

Features Pump

- › Fully automated pump that adapts individually
- › Large colour touch display with user-friendly interface
- › 3 modes: Alternating, Pulsating and Static
- › Pack&Go function – a setting which sucks air, deflating the mattress quickly and easily so it can be rolled
- › Silent running and vibration free
- › Battery backup (optional)
- › Soft friction hangers with extra soft tip
- › Notification function for loss of air and or electricity supply

Features Mattress

- › Soft, elastic hygienic cover that is vapour permeable
- › Easy to clean with well documented hygiene standard
- › Cell on cell construction with safety mattress
- › Heel function
- › Static head cells
- › CPR-function for rapid deflation
- › Smart cable management
- › PVC free
- › Unique design for easy handling and usage
- › Chlorine resistant (10%) multi stretch cover with welded seams
- › Well protected zipper

Specifications

Height	20cm
Function	Prevention and treatment up to and including category 4 Mod. Norton < 20 p Braden scale >12 p Waterlow scale 20+p
Recommended user weight	0-250kg
Input Voltage	100-240V / 50-60Hz
Sound level, pump, max	20dBA
Cleaning of cover	Wipe with cleaning agent and/or disinfectants Machine wash max 95°C, tumble drying

Award-winning technology.
Care of Sweden's fully
automatic products have been
internationally recognised for
their innovative engineering and
excellent performance.



 care of sweden

CuroCell Cirrus IQ

Pain therapy pressure care system

CuroCell® IQ Cirrus combines air with a soft foam surface and provides high user comfort and pressure redistribution. The heel function offers additional protection by relieving pressure on the heels. Moreover, the dual cover system reduces the shearing effect when adjusting the bed.

Benefits of the pulsation mode

We have made the CuroCell®IQ our most intelligent, yet simplest control unit so far. The Pulsating mode is the only featured programme, making it easier than ever to individualise effective care. The clinical results of the Pulsating mode has been proven in comprehensive studies, showing it to be superior to the traditional Alternating mode.

Pressure redistribution and pain therapy

CuroCell® IQ Cirrus is used as an aid in the prevention and treatment of pressure ulcers/pressure injuries up to and including category 4, and unclassified pressure ulcer, and for pain therapy. It is a replacement system ideal for all healthcare environments.

ORDER CODES:

Please contact us for order codes



Unique design for superior comfort

The unique design of CuroCell® Cirrus, providing a combination of air and foam, makes it possible to combine effective pressure redistribution and great comfort. The foam core surrounding the air cells provides stability and contributes to a smoother airflow, which is gentle for the user. With a liquid proof, yet vapour permeable cover, the mattress also reduces the risk of skin maceration.

Evac cover is an addition which enables a fast and secure evacuation. The soft inner cover is made of PU coating, an elastic fabric which reduces shearing and friction, and also provides good cleaning properties. The inner cover not only relieves shearing, but also contributes to the high comfort of the mattress. Reinforced sidewalls offer stability and safety for the user.



 care of sweden

CuroCell Cirrus A4

Pain therapy pressure care system

The CuroCell Cirrus 2.0 is used as an aid in the prevention and treatment of pressure ulcers up to category 4 (0-180 kg) and category 2 (181-230 kg), and for pain therapy. It is a replacement system ideal for all healthcare environments. CuroCell Cirrus 2.0 combines air with a soft foam surface and provides high user comfort and pressure redistribution.

The CuroCell® A4 Cirrus mattress system includes an intelligent control unit with a user-friendly interface. It is fully automatic without need for manual settings. The system identifies the user and automatically manages all settings to optimise the pressure according to the user's needs.

ORDER CODES:

0401-1740: 85 x 200x21cm

0401-1741: 105 x 200x21cm

0401-1742: 120 x 200x21cm

Features & Benefits

Mattress Features:

- › Unique wave like alternating process is particularly suitable for pain therapy
- › Added heel function offers even better protection by relieving pressure on the heels
- › Dual cover system reduces shearing effect when adjusting the bed
- › Hygienic cover with welded seams
- › Easy handling and cleaning
- › Soft, elastic, vapour permeable hygienic cover
- › Unique CPR-function with transport function

Pump Features:

- › Fully automated control unit that adapts individually
- › User friendly interface
- › Audible and visual notifications
- › Three program modes: Pulsating, Alternating and CLP (constant low pressure)
- › Caring mode (max-firm)
- › Pack&Go® function
- › Silent running and vibration free
- › Low voltage (12V power supply)
- › Soft friction hangers with extra soft tip
- › Air flow control; Constant temperature, increased user comfort, energy reduction





 care of sweden

CuroCell® IQ

Intelligent simplicity

The CuroCell UNO is used as an aid in the prevention and treatment of pressure ulcers up to and including category 4. It is equipped with a large touch screen panel with a user-friendly interface. The fully automated pump offers automatic adjustment, providing optimal pressure relief and comfort for each individual user while aiding simple use.

ORDER CODES:

Mattress Overlay

0401-1698: IQ CX10 80x200x10cm
 0401-1699: IQ CX10 85x200x10cm
 0401-1700: IQ CX10 90x200x10cm
 0401-1701: IQ CX10 100x200x10cm
 0401-1702: IQ CX10 105x200x10cm
 0401-1703: IQ CX10 120x200x10cm
 0401-1704: IQ CX15 80x200x15cm
 0401-1705: IQ CX15 85x200x15cm
 0401-1706: IQ CX15 90x200x15cm
 0401-1707: IQ CX15 100x200x15cm
 0401-1708: IQ CX15 105x200x15cm
 0401-1709: IQ CX15 120x200x15cm

Full Replacement

0401-1710: IQ CX16 80x200x16cm
 0401-1711: IQ CX16 85x200x16cm
 0401-1712: IQ CX16 90x200x16cm
 0401-1713: IQ CX16 100x200x16cm
 0401-1714: IQ CX16 105x200x16cm
 0401-1715: IQ CX16 120x200x16cm
 0401-1721: IQ CX20 80x200x20cm
 0401-1722: IQ CX20 85x200x20cm
 0401-1723: IQ CX20 90x200x20cm
 0401-1724: IQ CX20 100x200x20cm
 0401-1725: IQ CX20 105x200x20cm
 0401-1726: IQ CX20 120x200x20cm



Features & Benefits

Features Pump

- › Fully automated pump that adapts individually
- › Large colour touch display with user-friendly interface
- › 3 modes: Alternating, Pulsating and Static
- › Pack&Go function – a setting which sucks air, deflating the mattress quickly and easily so it can be rolled
- › Silent running and vibration free
- › Battery backup (optional)
- › Soft friction hangers with extra soft tip
- › Notification function for loss of air and or electricity supply

Features Mattress


- › Soft, elastic hygienic cover that is vapour permeable
- › Easy to clean with well documented hygiene standard
- › Cell on cell construction with safety mattress
- › Heel function
- › Static head cells
- › CPR-function for rapid deflation
- › Smart cable management
- › PVC free
- › Unique design for easy handling and usage
- › Chlorine resistant (10%) multi stretch cover with welded seams
- › Well protected zipper



Air Flow Control™

Air Flow Control

Constant temperature, lower noise levels, increased user comfort, energy reduction

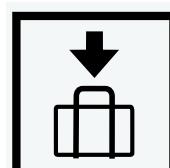

Pack&Go® simplifies deflation

With the Pack&Go® function, a press of a button is all that's needed to empty the mattress of air. Staff can easily pack the mattress away, and avoid having to manually squeeze the air out of used and dirty products.






Air Flow Control™

Air Flow Control

Constant temperature, lower noise levels, increased user comfort, energy reduction

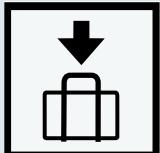

Pack&Go® simplifies deflation

With the Pack&Go® function, a press of a button is all that's needed to empty the mattress of air. Staff can easily pack the mattress away, and avoid having to manually squeeze the air out of used and dirty products.



Curocell A4

A fully automatic and flexible alternating pressure mattress system (overlay and replacement)

CuroCell A4 mattress is a new, fully automated alternating pressure system that's designed to provide excellent comfort and save time and resources in working to prevent pressure injuries up to category 4.

The CuroCell A4 increases comfort for the user, enables better health economy and leads to an easier working day for care staff.

ORDER CODES:

0401-1600: A4 OP10 85x200x10cm

0401-1601: A4 OP10 90x200x10cm

0401-1630: A4 CX10 85x200x10cm

0401-1631: A4 CX10 90x200x10cm

0401-1632: A4 CX10 100x200x10cm

0401-1633: A4 CX10 105x200x10cm

0401-1634: A4 CX10 120x200x10cm

0401-1728: A4 CX15 80x200x15cm

0401-1729: A4 CX15 85x200x15cm

0401-1730: A4 CX15 90x200x15cm

0401-1731: A4 CX15 100x200x15cm

0401-1732: A4 CX15 105x200x15cm

0401-1733: A4 CX15 120x200x15cm

0401-1660: A4 CX16 85x200x16cm

0401-1661: A4 CX16 90x200x16cm

0401-1662: A4 CX16 100x200x16cm

0401-1663: A4 CX16 105x200x16cm

0401-1664: A4 CX16 120x200x16cm

0401-1690: A4 CX20 85x200x20cm

0401-1691: A4 CX20 90x200x20cm

0401-1692: A4 CX20 100x200x20cm

0401-1693: A4 CX20 105x200x20cm

0401-1694: A4 CX20 120x200x20cm

Features & Benefits

➤ CuroCell A4 is available in five variants:

CuroCell A4 OP10: 10 cm overlay mattress up to 160kg

CuroCell A4 CX10: 10 cm overlay mattress up to 200kg

CuroCell A4 CX15: 15 cm overlay mattress up to 200kg

CuroCell A4 CX16: 16 cm replacement mattress up to 200kg

CuroCell A4 CX20: 20 cm replacement mattress up to 250kg

➤ The CuroCell A4 mattress system has three program modes: Pulsation, Alternating and CLP (constant low pressure/static)

➤ With the new pulsation program, the pressure in the mattress cells alternates between two-thirds and one-third pressure, the idea being to provide gentler pressure relief for the user. CuroCell A4 is delivered with pulsation mode as the default setting. CuroCell A4 also has alternating and CLP (constant low pressure) program modes

➤ The pulsation mode is highly efficient and appreciated by users. Based on positive feedback as well as relevant research, the pulsation mode is the default setting. Choice of mode is simply performed by the touch of a button and should always be based on the user's status


➤ All the mattresses are made of materials that meet the healthcare sector's demands on hygiene, safety and quality. The mattresses are designed to allow the user excellent comfort, minimize the effects of shearing forces and provide a good microclimate. The polyurethane air cells entails lower noise levels and greater comfort. The air in the cells is reused in the alternations, thus reducing electricity consumption and noise, and improving micro climate for the user. If the mattress should temporarily lose its power supply, the air remains inside

➤ The pump automatically optimizes the settings for the user of the mattress. No manual setting is required. The engineering further reduce noise levels, and cause less movement in the mattress. The result is a quiet, pleasant environment with a high level of comfort, where the system itself is barely noticeable


Pulsation



Alternating

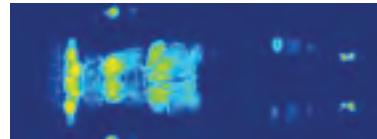

CLP

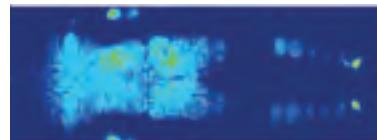

Specifications

CuroCell A4 OP10 10cm overlay mattress up to 160kg

CuroCell A4 CX10 10cm overlay mattress up to 200kg

CuroCell A4 CX15 15cm overlay mattress up to 200kg

CuroCell A4 CX16 16cm replacement mattress up to 200kg

CuroCell A4 CX20 20cm replacement mattress up to 250kg



 care of sweden

CuroCell S.A.M. Overlay

Mattress overlay system with very high comfort

The CuroCell S.A.M. Overlay is used as an aid in the prevention and treatment of pressure ulcers up to category 2, and for pain therapy. It is a static air mattress overlay system that provides very good comfort and pressure redistribution.

ORDER CODES:

0401-1506: 80 x 200 x 10cm
 0401-1507: 85 x 200 x 10cm
 0401-1508: 90 x 200 x 10cm
 0401-1509: 100 x 200 x 10cm
 0401-1510: 105 x 200 x 10cm
 0401-1511: 120 x 200 x 10cm

Features & Benefits

- › CPR function – rapid deflation
- › Easy to handle and maintain
- › Silent and energy-efficient pump
- › Soft, elastic, vapour permeable hygienic cover
- › Very good cleaning properties
- › Made from modern, recyclable materials and meets health care requirements for hygiene, quality and safety

Specifications

Height	10cm
Function	Prevention and treatment up to and including category 2 Mod. Norton < 20 p Braden scale 13-14 p Waterlow scale > 10+ p
Recommended User Weight	0-200kg
Power Supply	230V / 50Hz
Cleaning of Cover	Wipe with cleaning agent and/or disinfectants Machine was max 95°C, tumble drying



 care of sweden

CuroCell S.A.M. CF16

Mattress replacement system with very high comfort

The CuroCell S.A.M. CF16 is used as an aid in the prevention and treatment of pressure ulcers up to category 2, and for pain therapy. It is a static air mattress which provides very good comfort and pressure relief. It is a mattress replacement system with an integrated foam core.

ORDER CODES:

0401-1500: 80 x 200 x 16cm
0401-1502: 90 x 200 x 16cm
0401-1503: 100 x 200 x 16cm
0401-1504: 105 x 200 x 16cm
0401-1505: 120 x 200 x 16cm

Features & Benefits

- > Extra soft inner cover meaning already low pressure is further reduced
- > CPR function – rapid deflation
- > Easy to handle and maintain
- > Heel function
- > Silent and energy-efficient pump
- > Soft, elastic, vapour permeable hygienic cover
- > Very good cleaning properties

Specifications

Height	16cm
Function	Prevention and treatment up to and including category 2 Mod. Norton < 20 p Braden scale 13-14 p Waterlow scale > 10+ p
Recommended User Weight	0-200kg
Power Supply	230V/50Hz
Cleaning of Cover	Wipe with cleaning agent and/or disinfectants Machine wash max 95°C, tumble drying



 care of sweden

CuroCell AREA Zone

Functional mattress replacement system with self-adjusting technology

CuroCell AREA Zone is a functional mattress replacement system with self-adjusting technology, used as an aid in the prevention and treatment of pressure ulcers up to category 3. It combines air with foam for effective pressure redistribution and great comfort. The mattress is self-adjusting and non-powered. It is a flexible mattress replacement system that can be used in all kinds of healthcare settings, easy to use and handle.

ORDER CODES:

0401-1518: 85 x 200 x 18cm
 0401-1519: 90 x 200 x 18cm
 0401-1520: 100 x 200 x 18cm
 0401-1521: 105 x 200 x 18cm
 0401-1522: 120 x 200 x 18cm

Specifications

Height	18cm
Function	Prevention and treatment up to and including category 3 Mod. Norton < 20 p Braden scale 13-14 p Waterlow scale > 10+ p
Recommended User Weight	0-230kg
Cleaning of Cover	Wipe with cleaning agent and/or disinfectants Machine wash max 95°C, tumble drying

Features & Benefits

- › Combines foam with air for effective pressure redistribution
- › Nine air cells that are divided into zones which adapt automatically to the user
- › Built up with multiple layers of high quality foam
- › Special design reduces shear forces
- › Built-in sloped heel function to reduce pressure on the heel
- › Elastic and flexible removable cover with well-documented standard of hygiene
- › Welded seams
- › Soft and elastic material is vapour permeable which reduces risk of skin maceration
- › Rated "High pressure relief" in external tests
- › When tested, the mattress' capacity of pressure redistribution (PAI) it is reported that 100 % of the pressure on the mattress is below the threshold of 30mmHg (load 80kg)
- › Evacuation cover available on request



 care of sweden

Optimal 5zon

The complete mattress for pressure ulcer prevention

The Optimal 5zon is the complete mattress for pressure ulcer prevention. Ideal for all types of healthcare environments as an aid in the prevention and treatment of pressure ulcers up to category 2 and for pain therapy.

ORDER CODES:

0402-2000: 85 x 200 x 14cm
0402-2001: 90 x 200 x 14cm
0402-2002: 100 x 200 x 14cm
0402-2003: 105 x 200 x 14cm
0402-2004: 120 x 200 x 14cm

Features & Benefits

- › **Pressure**
Highly elastic, high-quality material and a unique zone and module system gives effective pressure equalisation
- › **Shear forces**
The mattress is specially designed to minimise the effects of shear forces
- › **Sit function**
Patented sit function DuoCore is designed to give support when sitting and also to provide good comfort when lying
- › **Moisture**
The hygienic cover is vapour permeable, which reduces the risk of skin maceration
- › **Hygiene**
The covers are easy to clean and handle and has a very well documented hygienic standard
- › Evacuation cover available on request

Specifications

Core	HR foam 65+50kg/m ³
Height	14cm
Function	Prevention and treatment up to and including category 2 Mod. Norton < 20 p Braden scale 13-14 p Waterlow scale > 10+ p
Recommended User Weight	100-300kg
Cleaning of Cover	Wipe with cleaning agent and/or disinfectants Machine wash max 95°C, tumble drying



 care of sweden

Optimal 5zon Plus

Preventative mattress replacement for bariatric use

The Optimal 5zon Plus is used as an aid in the prevention and treatment of pressure ulcers up to category 2, and is ideal for all healthcare environments. It is a mattress replacement that has been specially designed for bariatric use.

ORDER CODES:

0402-2010: 85 x 200 x 15cm

0402-2011: 90 x 200 x 15cm

0402-2012: 100 x 200 x 15cm

0402-2013: 105 x 200 x 15cm

0402-2014: 120 x 200 x 15cm

Features & Benefits

- › **User weight**
For bariatric use, recommended user weight is 100-270kg
- › **Pressure**
Highly elastic, high-quality material and a unique zone and module system gives effective pressure equalisation
- › **Shear forces**
The mattress is specially designed to minimise the effects of shear forces
- › **Sit function**
Patented sit function DuoCore is designed to give support when sitting and also to provide very good comfort when lying
- › **Moisture**
The hygienic cover is vapour permeable, which reduces the risk of skin maceration
- › **Hygiene**
The cover is easy to clean and has a very well documented hygienic standard

Specifications

Core	HR foam 65+50kg/m ³
Height	15cm
Function	Prevention and treatment up to and including category 2 Mod. Norton < 20 p Braden scale 13-14 p Waterlow scale > 10+ p
Recommended User Weight	100-270kg
Cleaning of Cover	Wipe with cleaning agent and/or disinfectants Machine was max 95°C, tumble drying



 care of sweden

Optimal Cura Mattress

Preventative mattress replacement with good pressure equalization

Optimal Cura is used as an aid in the prevention and treatment of pressure ulcers up to category 1. It is a mattress replacement, used in all types of healthcare environments.

ORDER CODES:

0401-1586: 85 x 200cm

0401-1587: 90 x 200cm

0401-1588: 100 x 200cm

0401-1589: 105 x 200cm

0401-1590: 120 x 200cm

Features & Benefits

- Optimal Cura is a two layer mattress where the top layer is made from visco-elastic foam (so called memory foam) and the bottom layer is made from HR foam. The special design of the mattress and the materials gives very good pressure equalization. The two layers allows the body to sink into the material, and thereby lower the pressure against the body
- **Pressure**
Highly elastic, high-quality material and an unique design with effective pressure equalisation
- **Shear forces**
The mattress is specially designed to minimise the effects of shear forces
- **Moisture**
The hygienic cover is vapour permeable, which reduces the risk of skin maceration
- **Hygiene**
The cover is easy to clean and has a very well documented hygienic standard

Specifications

Core	HR foam + visco-elastic foam 50kg/m3
Height	13cm
Covers	Olivia, Olivia H, Olivia/Evac, Nordic and Steel/Evac
Function	Prevention & treatment up to and including category 1
Recommended User Weight	0-130kg
Fire	EN 597-1, -2
Cleaning of Cover	Wipe with cleaning agent and/or disinfectants Machine wash max 95°C, tumble drying



 care of sweden

Optimal Care Mattress

Quality pressure redistribution mattress with excellent comfort

Optimal Care Mattress is a pressure redistributing mattress divided into zones adapted to the body. The Optimal Care is a quality mattress designed to meet stringest requirements in terms of quality, comfort and support.

ORDER CODES:

0401-1576: 85 x 200 x 14cm

0401-1577: 90 x 200 x 14cm

0401-1578: 100 x 200 x 14cm

0401-1579: 105 x 200 x 14cm

0401-1580: 120 x 200 x 14cm

Features & Benefits

- › Reinforced sides to give support when getting into and out of bed
- › The unique label contains the necessary information to ensure safe and easy use of this mattress
- › **Pressure**
Highly elastic, high-quality material with a unique module system gives effective pressure equalisation
- › **Shear forces**
The mattress is specially designed to minimise the effects of shear forces
- › **Sit function**
Patented sit function DuoCore is designed to give support when sitting and also to provide good comfort when lying
- › **Moisture**
The hygienic cover is vapour permeable(3), which reduces the risk of skin maceration
- › **Hygiene**
The cover is easy to clean and has a very well documented hygienic standard

Specifications

Core	Highly elastic cold foam 50kg/m3
Height	14cm
Covers	Olivia/Evac, Olivia, Olivia H, Nordic and Steel/Evac
Function	Prevention & treatment up to and including category 2(1)
Recommended User Weight	0-140kg
Fire	EN 597-1, -2
Cleaning of Cover	Wipe with cleaning agent and/or disinfectants Machine wash max 95°C, tumble drying



care of sweden

Curera Support Wedge

The Curera Support Wedge is perfect for positioning and position changes. It can be used to place a patient/user in a side position in order to relieve the sacrum, an extra sensitive area. For the side position, two support wedges are used together. The Curera Support Wedge is easy to use and clean. It comes with a liquid resistant cover that can be wiped with cleaning agents and/or disinfectants. It can also be machine washed and is sold in sets of two wedges.



ORDER CODE: 0404-0401

Features & Benefits

- › Enables repositioning of patients by tilting the side of mattress without touching the patient. Ideal where moving the patient causes pain
- › Quickly and easily achieve the 30° tilt
- › Helps prevent pressure ulcers
- › Liquid resistant cover can be wiped with a cleaning agent and/or disinfectants. It can also be machine washed
- › Examples of different uses:
When the user has problems lying on their side
For relief and prevention of pressure ulcers
When the user has problems with position changes
As a kick protection against bed rails
For other positioning



Specifications

Filling	100% foam
Cover	Polyurethane / polyester
Size	70 x 26 x 15/5 cm
Cleaning of cover	Wipe with cleaning agent and / or disinfectant Machine wash 95°C, tumble drying



Levabo Pressure Relief Aids

The Levabo Pressure Relief Positioning Aids are a single patient use pressure care solution.

These innovative products were developed in collaboration with leading wound care nurses and are manufactured in Denmark.

The products have a series of advantages compared to conventional positioning products. With the use of Levabo's positioning cushions, you always have the needed cushion at hand for effective prevention of pressure sores.

The cushions are inflatable, multi functional and single patient use which can be disposed after use.



For more information on this range, visit our website activehealthcare.co.nz




ACTION

Action Shear Smart Pad

The Action Shear Smart Pad, combined with the breathable double-layer Shear Smart cover significantly reduces the effects of shear while providing superb user comfort. The cube construction of the pad surface allows air circulation, provides cushioning from mild impact, heat dissipation, and the comfort of sitting on 9/16" Akton polymer. This cushion will conform to contoured bases for additional support when required.

ORDER CODES:

- 0404-1001: 16" x 16"
- 0404-1002: 16" x 18"
- 0404-1003: 18" x 16"
- 0404-1004: 18" x 18"
- 0404-1005: 18" x 20"
- 0404-1006: 20" x 18"
- 0404-1007: 20" x 20"

Features & Benefits

- › Akton polymer relieves pressure, reduces shear forces, manages heat & humidity build-up and is 100% hassle-free
- › Its unique chemical structure means you don't have to worry about leaks, cuts, punctures, oozing, deflating or bottoming out
- › Enjoy peace of mind knowing this ultra-soft polymer provides comfortable, full-memory support that's proven clinically effective and long-lasting durability

Specifications

Sizes	16" x 16", 16" x 18", 18" x 16", 18" x 18", 18" x 20", 20" x 18", 20" x 20"
Height	9/16" (14mm)
Weight	2.8kg
Weight limit	102kg
Accessories	Shear Smart cover

ACTION

Action Commuter Cushion

The Action Commuter Cushion offers basic pressure and shear reduction for low risk users, at an affordable price. Combined ½" Akton polymer and 1 ¾" foam are sealed in a waterproof film barrier to guard against incontinence permeation.

ORDER CODES:

- 0404-1100: 16" x 16"
- 0404-1103: 18" x 16"
- 0404-1106: 18" x 18"
- 0404-1109: 16" x 18"
- 0404-1112: 18" x 20"
- 0404-1115: 20" x 18"
- 0404-1118: 20" x 20"

Features & Benefits

- › Lightweight and portable design
- › Pressure relief and shear reduction
- › Akton polymer topper with foam base
- › Waterproof film for incontinence protection
- › Comes with a basic cover
- › Affordable price

Specifications

Sizes	16" x 16", 16" x 18", 18" x 16", 18" x 18", 18" x 20", 20" x 18", 20" x 20"
Height	2 1/4" (57mm)
Weight limit	102kg
Accessories	Basic cover included



ACTION®

Action Centurian Cushion

The Action Centurian Cushion provides pressure relief and shear reduction for the low to moderate risk user. The Action Centurian offers significant comfort for the active client who needs a light to medium weight cushion with low maintenance.

ORDER CODES:

- 0404-1145: 16" x 16"
- 0404-1148: 18" x 16"
- 0404-1151: 16" x 18"
- 0404-1154: 18" x 18"
- 0404-1157: 18" x 20"
- 0404-1160: 20" x 18"
- 0404-1163: 20" x 20"

Features & Benefits

- › Combined layers of 5/8" Akton polymer topper and 5/8" foam base
- › Sealed in waterproof film for incontinence protection
- › Pressure relief and shear reduction for the low to moderate risk user
- › Offers significant comfort for the active client who needs a light to medium weight cushion with low maintenance
- › Comes with a basic cover

Specifications

Sizes	16" x 16", 16" x 18", 18" x 16", 18" x 18", 18" x 20", 20" x 18", 20" x 20"
Height	1 1/4" (32mm)
Weight	3.3kg
Weight limit	102kg
Accessories	Basic cover included. Incontinence cover available

ACTION®

Action Pilot Cushion

The Action Pilot Cushion is a low-profile cushion that provides exceptional pressure relief for most users with a pressure ulcer history. 1" of Akton polymer provides immersion for bony prominences and protects clients with fragile skin integrity. This multi-use cushion is recommended for use on manual and power wheelchairs, truck or taxi seats, or for other uses requiring significantly reduced high pressure areas, vibration and shear.

ORDER CODES:

- 0404-1121: 16" x 16"
- 0404-1124: 16" x 18"
- 0404-1127: 18" x 16"
- 0404-1130: 18" x 18"
- 0404-1133: 18" x 20"
- 0404-1136: 20" x 16"
- 0404-1139: 20" x 18"
- 0404-1142: 20" x 20"

Features & Benefits

- › Very low profile allowing users to effectively reach the floor for more efficient foot-assisted propulsion
- › Conforms to pre-contoured foam or rigid cushions to maximize stability and positioning when required
- › 1" Akton polymer gel provides immersion for bony prominences and protects clients with fragile skin integrity
- › Comes with basic cover. Incontinent and replacement covers are also available

Specifications

Sizes	16" x 16", 16" x 18", 18" x 16", 18" x 18", 18" x 20", 20" x 16", 20" x 18", 20" x 20"
Height	1" (25mm)
Weight	5.2kg
Weight limit	102kg
Accessories	Basic cover included. Incontinence cover available


ACTION

Action Professional Cushion

The Action Professional Cushion provides exceptional pressure relief for users at risk of skin breakdown, or with a history of pressure ulcers. A full 1 5/8" thickness of Akton polymer provides effective weight distribution on a contoured or flat surface. The Professional provides an additional 50% cushioning against pressure, shear and vibration. This is significant for active clients using a power wheelchair who have concerns about bottoming-out and the potential effects of increased temperatures (due to foam insulation properties).

ORDER CODES:

- 0404-1166: 16" x 16"
- 0404-1169: 18" x 16"
- 0404-1172: 16" x 18"
- 0404-1175: 18" x 18"
- 0404-1178: 18" x 20"
- 0404-1181: 20" x 18"
- 0404-1184: 20" x 20"

Features & Benefits

- › 1 5/8" Akton Polymer gel cushion
- › Provides effective weight distribution for both flat and contoured surfaces
- › Suitable for bariatric users
- › Comes with Basic Cover. Incontinent and replacement covers are also available

Specifications

Sizes	16" x 16", 16" x 18", 18" x 16", 18" x 18", 18" x 20", 20" x 18", 20" x 20"
Height	1 5/8" (41mm)
Weight	8.4kg
Weight limit	222kg
Accessories	Basic cover included. Incontinence cover available

ACTION

Action Xact Classic Cushion

The Action Xact Classic Cushion offers outstanding pressure distribution and anti-shearing properties. The Action Cube Pad and laminated foam base help the user maintain a comfortable, therapeutic position. Rehab therapists most often recommend the Action Xact Classic for users requiring forward and lateral positional support. The recent re-design that incorporates viscoelastic pink foam offers improved support for gluteal areas and bony protuberances.

ORDER CODES:

- | | |
|--------------------|--------------------|
| 0404-1035: 10"X10" | 0404-1062: 16"X16" |
| 0404-1038: 10"X12" | 0404-1065: 16"X18" |
| 0404-1041: 12"X10" | 0404-1068: 16"X20" |
| 0404-1044: 12"X12" | 0404-1071: 17"X17" |
| 0404-1047: 12"X14" | 0404-1077: 18"X18" |
| 0404-1050: 14"X12" | 0404-1080: 18"X20" |
| 0404-1053: 14"X14" | 0404-1083: 20"X16" |
| 0404-1056: 14"X16" | 0404-1086: 20"X18" |
| 0404-1059: 16"X14" | 0404-1089: 20"X20" |

Features & Benefits

- › Contoured foam base
- › Visco memory foam ischial insert
- › 9/16" cube Akton polymer topper
- › Paediatric sizes available
- › Suitable for bariatric users

Specifications

Sizes	10"x10", 10"x12", 12"x10", 12"x12", 12"x14", 14"x12", 14"x14", 14"x16", 16"x14", 16"x16", 16"x18", 16"x20", 17"x17", 18"x16", 18"x18", 18"x20", 20"x16", 20"x18", 20"x20"
Height	2.5" – 4"
Weight	From 1.2kg
Weight limit	178kg
Accessories	Body smart cover included. Incontinence cover available



ACTION

Action Xact Lite Cushion

The Action Xact Lite Cushion introduces a new technology in lightweight cushions. At 1.45kg the Xact Lite offers the active user excellent pressure reduction and exceptional anti-shearing properties. The Akton polymer-textured overlay, on a contoured multi-layered foam base, promotes therapeutic positioning and comfort.

ORDER CODES:

- 0404-1209: 16" x 16"
- 0404-1210: 16" x 18"
- 0404-1211: 16" x 20"
- 0404-1212: 18" x 16"
- 0404-1213: 18" x 18"
- 0404-1214: 18" x 20"
- 0404-1215: 20" x 16"
- 0404-1216: 20" x 18"
- 0404-1217: 20" x 20"

Features & Benefits

- > Extremely low weight, low profile design
- > 2 3/4" lightweight contoured multilayered foam base
- > 3/16" pebbled Akton polymer gel topper

Specifications

Sizes	16"x16", 16"x18", 16"x20", 18"x16", 18"x18", 18"x20", 20"x16", 20"x18", 20"x20"
Height	3" (76mm)
Weight	1.45kg
Weight limit	114kg
Accessories	Basic cover included. Incontinence cover available

ACTION

Action Xact LP2 Cushion

The Action Xact LP2 Cushion allows low seat to floor height for active users and moderate-to-high risk clients. Its unique construction floats the user in a pool of Action polymer, cushioning critical pressure points for long lasting comfort and support. The innovative cover provides minimal shear stress by preventing forward migration. Only 2" in height.

ORDER CODES:

- 0404-1218: 16" x 16"
- 0404-1219: 16" x 18"
- 0404-1220: 16" x 20"
- 0404-1221: 18" x 16"
- 0404-1222: 18" x 18"
- 0404-1223: 18" x 20"
- 0404-1224: 20" x 16"
- 0404-1225: 20" x 18"
- 0404-1226: 20" x 20"

Features & Benefits

- > Akton polymer top layer offers low shear comfort, temperature, humidity control and constant protection from pressure sores
- > High resilient base with back wedges promote stable pelvic positioning, support trochanters and decreased pressure at the ischial tuberosities
- > Polymer and foam sacral reservoir creates added support and prevents bottoming out
- > Contouring provides pelvic and lower limb positioning as well as offloading for high risk areas
- > 2" height allows easy leg propulsion
- > Breathable spacer fabric cover
- > Protective incontinent resistant internal cover
- > Weight capacity 300lbs

Specifications

Sizes	16"x16", 16"x18", 16"x20", 18"x16", 18"x18", 18"x20", 20"x16", 20"x18", 20"x20"
Height	2" (51mm)
Weight	2.25kg
Weight Limit	136kg

Accessories Dual layer spacer cover and Incontinence resistant internal cover included



ACTION[®]

Action Xact Soft Cushion

The Action Xact Soft Cushion is recommended for moderate to high risk users and clients requiring pressure relief and support. 3 1/2" countered multi-layer foam base with 9/16" cube Akton cube polymer insert.

ORDER CODES

0404-1227: 16" x 16"
 0404-1228: 16" x 18"
 0404-1229: 16" x 20"
 0404-1230: 18" x 16"
 0404-1231: 18" x 18"
 0404-1232: 18" x 20"
 0404-1233: 20" x 16"
 0404-1234: 20" x 18"
 0404-1235: 20" x 20"

Features & Benefits

- › The Xact Soft offers outstanding comfort, pressure distribution and immersion
- › The 9/16" Action Cube Pad embedded in a laminated foam base assists the user to maintain a comfortable, supportive position
- › Cushion design includes a pommel and lateral thigh supports to encourage lower extremity positioning and hip alignment, combined with a pre-ischial shelf to minimise sliding
- › Recommended for moderate to high risk users and clients requiring pressure relief and forward and lateral positional support
- › Comes with a cover. Incontinent and replacement covers are also available

Specifications

Sizes	16"x16", 16"x18", 16"x20", 18"x16", 18"x18", 18"x20", 20"x16", 20"x18", 20"x20"
Height	4" (102mm)
Weight	1.7kg
Weight Limit	160kg
Accessories	Body Smart cover included. Incontinence cover available